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# User instructions Microlens Soft contact lenses

microlens  
you will see

Ophtecs Europe b.v.  
Marithaime 20 | 6662 WD Elst

## Description

The Microlens soft contact lenses are soft hydrophilic contact lenses designed for daily use and three-monthly or semi-annual replacement. The replacement schedule and the type of contact lens solution are determined by your contact lens specialist / optometrist / ophthalmologist.

## Material

The material is described on the label of the packaging.

## Packaging

The lenses are delivered in separate sealed glass or plastic vials. Lens power, base curve radius, diameter and expiring date are described on the label. Don't use the lens if the packaging is broken or the expiring date has passed.

## Indications

Microlens Soft contact lenses are intended for the correction of visual acuity defects.

## Contra-indications

- Inflammations of the anterior eye segment.
- Eye diseases that affect the cornea or conjunctiva.
- Insufficient tear secretion.
- Systemic diseases that affect the eye.
- When Soft contact lenses cause or increase eye allergies.
- Medication with a contra-indication for Soft contact lenses. (See the instructions for use before taking the medication)

## Fitting and wearing instruction

The Microlens soft contact lenses are only fitted, checked and delivered by qualified optometrist / contact lens specialist / ophthalmologist.

## Aftercare

The eyes and contact lens fit should be regularly checked according to the instructions of your optometrist / contact lens specialist / ophthalmologist.

## Contact lens insertion

1. Always wash and dry your hands before touching your lenses.
2. Make sure the lenses aren't turned inside out: Place the lens on your index finger to see if it has the correct shape. (Fig. 1 & 2)
3. Use the index finger and middle finger to spread the eyelids. Place the lens on the eye. (Fig. 3).
4. Repeat the procedure for the other eye.

## Contact lens removal

1. Always wash and dry your hands before touching your lenses.
2. Look upwards and hold down the lower eyelid.
3. Gently squeeze the edge of the contact lens with the tip of your forefinger and thumb and remove the lens from your eye. (Fig. 4)
4. Repeat the procedure for the other eye.

Fig. 1

Fig. 2

Fig. 3

Fig. 4

## Possible side effects

In the case of the following sensations:

- burning, stinging or itching eyes;
- reduced wear time;
- increased sensitivity to light;
- excessive tearing, eye discharge or abnormal redness of the eyes;
- reduced visual acuity; immediately contact your optometrist / contact lens specialist / ophthalmologist.

## Contact lens cleaning

Always follow the user instructions of prescribed contact lens solutions of your optometrist / contact lens specialist / ophthalmologist.

## Lens care

- The Microlens soft contact lenses are for daily use only and should not be worn during sleep.
- Always wash your hands before touching the contact lens.
- Do not let the contact lens come into contact with soap, lotion, deodorant or any other body product because this can cause irritation of the eye when inserting the lens.
- Avoid using aerosol products like hairspray or paint spray during lens wearing. If sprays are used wear safety goggles or close the eyes.
- Never use sharp objects or long fingernails to handle the soft contact lenses.
- Carefully follow the instructions on lens solution use by your optometrist / contact lens specialist / ophthalmologist
- Always use daily refreshed lens solutions, never use tap water or saliva.
- Inform your optometrist / contact lens specialist / ophthalmologist about the use of medication which may affect successful contact lens wear.

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