ENGLISH | Instructions for Use

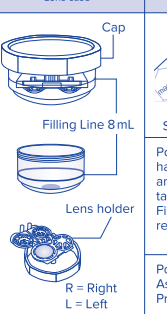
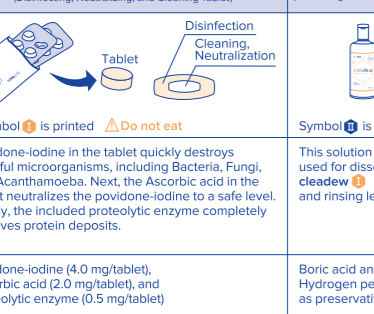
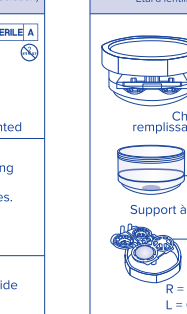
Disinfecting, Cleaning and Storing System

Contact lens disinfection system which disinfects, neutralizes, and removes protein deposits for all soft contact lenses including silicone hydrogel lenses.

Read this instruction sheet carefully before using **cleadow**, and keep it for future reference. To avoid eye infections or damage, always follow the user instructions.

COMPOSITION AND DESCRIPTION

Do not use **cleadow** if allergic to any of the ingredients used in **cleadow**.

| cleadow | cleadow I | cleadow II |
|--|--|---|
| Lens case (Disinfecting, Neutralizing, and Cleaning Solution) | (Disinfecting, Neutralizing, and Cleaning Tablet) | (Dissolving & Rinsing Solution) |
|  Filling Line 8 mL R = Right L = Left |  Symbol I is printed, Δ Do not eat Povidone-iodine in the tablet quickly destroys harmful microorganisms, including Bacteria, Fungi, and Acanthamoeba. Next, the Ascorbic acid in the tablet neutralizes the povidone-iodine to a safe level. Finally, the included proteolytic enzyme completely removes protein deposits. |  STERILE A Symbol II is printed This solution is used for dissolving cleadow I and rinsing lenses. |
| | Povidone-iodine (4.0 mg/tablet), Ascorbic acid (2.0 mg/tablet), and Proteolytic enzyme (0.5 mg/tablet) | Boric acid and Hydrogen peroxide as preservative |

USER INSTRUCTIONS

- 1  Open the lens holders and place each lens on each dome. Close the lens holders with care.
- 2  Add a tablet of **cleadow I** into the lens case and fill the case with **cleadow II** up to the filling line. Do not use tablet if discoloured or broken.
- 3  Soak the lenses for **at least 4 hours**. Cleaning process begins after the solution colour changes.
- 4  **Check colour!** Ensure that solution and lens are completely clear. If orange colour remains even after 4 hours, see **PRECAUTIONS** below.
- 5  Remove the lenses.
- 6  Rinse lenses in the palm of your hand with **cleadow II** before wearing them.

STORAGE

After completing the above steps, in case of storing the lens for more than 24 hours, you should replace the solution with **cleadow II** as a conservation solution. It is recommended to use the solution. You may store the lens in the unopened lens case up to 7 days, and then repeat the care steps from the beginning before wearing your lens.

WARNINGS

- Do not use **cleadow** if allergic to any of the ingredients used in **cleadow**.
- If you experience persistent eye discomfort, excessive tearing, vision changes, or redness of the eye, remove your lenses immediately and consult your eye doctor or an eye care practitioner.
- Do not use **cleadow** with any eye medication or other contact lens care products.

PRECAUTIONS

- Read this instruction sheet carefully before using **cleadow** and keep it for future reference.
- To avoid eye infections, always follow the user instructions.
- Always wash and dry your hands completely before handling your lenses.
- The solution does not turn orange after you have added a tablet, discard the solution, wash and dry the lens case, and then repeat the care steps from the beginning, in accordance with the user instructions.
- Do not wear your lenses if the orange stain persists.
- Consult your eye care practitioner if you have worn un-neutralized lenses.
- Ensure that solution and lens are completely clear before wearing the lenses. If orange colour remains after 4 hours, lightly shake the lens case and leave it aside for 20–30 minutes. Check if the colour has disappeared. If orange colour persists, discard solution and repeat care steps from the beginning.
- After each use, discard any **cleadow** solution that remains in the lens case.
- After each use of **cleadow**, wash the lens case with sterile saline solution and allow it to air dry.
- Children should not use **cleadow** by themselves; parents are requested to be responsible for their daily care.
- Ensure that solution and lens are completely clear before wearing the lenses. If orange colour remains after 4 hours, lightly shake the lens case and leave it aside for 20–30 minutes. Check if the colour has disappeared. If orange colour persists, discard solution and repeat care steps from the beginning.
- Do not replace **cleadow I** with any other container.
- Do not use **cleadow** with an old lens case.
- Always use the new lens case provided.
- Use the tablet immediately after opening the package.
- Always store the tablets in their aluminum package when not in use.
- Do not change the contact lens care system without consulting eye care professional.

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