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OPTHECS Europe B.V.  
Marithaime 20,  
6662 WD Elst, The Netherlands

## Possible side effects

In case of reduced comfort of the Microlens scleral lenses like:

- Burning, stinging or itchy eyes.
- Reduced wear time.
- Increased sensitivity to light.
- Excessive tears, eye discharge or abnormal redness of the eyes.
- Reduction of visual acuity.

If one of these effects occurs you should immediately contact your medical contact lens specialist / optometrist / ophthalmologist!

## Lens care

- First wash your hands before you touch the lens.
- Some eye irritations are known to be caused by a lens that comes into contact with soap, lotion or deodorant.
- Do not use sharp objects or long fingernails to retrieve the lens from case or container.
- Avoid using aerosol products like hairspray if you wear the lenses. If sprays are used, please close both eyes thoroughly.
- Carefully follow the instructions on lens solutions and wearing times as instructed by your eye care specialist.
- Always use fresh lens solutions.

## Description

The MISA and mini-MISA scleral lenses are lenses that are placed on the eye and rests on the white part (sclera) of the eye. The central part of the lens, that regulates the visual correction and restoration of the cornea, vault over the entire corneal surface without touching. Scleral lenses from Microlens are manufactured with highly automated ultra-precision according to the recipe of your medical contact lens specialist / optometrist / ophthalmologist.

## Material

Copolymer of Fluorine, Silicone and Methacrylates.

## Packaging

The MISA and mini-MISA scleral lenses are delivered in a dry state to your medical contact lens specialist / optometrist / ophthalmologist. To optimize the surface properties before insertion the lenses must be thoroughly treated with the proper insertion contact lens solutions by your medical contact lens specialist / optometrist / ophthalmologist. Only this will guarantee that adequate hydrophilic properties will be adapted by the lens.

## Indications

The MISA and mini-MISA scleral lenses are used for the following indications:

### To improve visual acuity:

- If the cornea is irregular or shows an abnormal topography
- After post-surgical corneal and refractive surgery
- In case of corneal degeneration and dystrophy

### To reduce pain and irritation in case of:

- Dry eye management: Stevens Johnson syndrome, Sjögren's Syndrome
- Ptosis
- Trichiasis

### To improve comfort and safety of lens wearing in case of:

- Work in very dusty conditions
- Contact- and extreme sports

## Fitting and wearing instructions

The MISA and mini-MISA scleral lenses are only fitted and dispatched by medical contact lens specialists / optometrists / ophthalmologists, being trained by Microlens.

## Aftercare

- Apart from the actual use of the Microlens scleral lenses your eyes have to be regularly checked up as instructed by your eye care specialist.
- Always follow the instructions on the use of contact lens solutions by your medical contact lens specialist / optometrist / ophthalmologist, and always use their prescribed type of solutions.
- Inform your eye care specialist on the drugs you use: they may affect successful contact lens wear.
- Never insert a visibly damaged lens.
- Always follow the prescribed maximum wearing time.

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